



# IV Glove Strapless

***Pre-application: Clean the site per hospital protocol, and ensure the site is dry before applying the***

**Instructions  
for use**


***For ages 12 and under***

# IV Glove

1. Follow standard IV insertion as per hospital protocol
2. Cover the insertion site with medical tape or Tegaderm
3. Open the transparent covered dome of the IV Glove by pulling up on the cover
4. Remove the plastic from the dual-sided adhesive strip on the bottom of the device
5. Place catheter tubing into patented S-Clip design and place the device on the patients hand, ensuring the dual-sided adhesive adheres to the skin to keep the device in place
6. Carefully close the protective dome, ensuring the pins 'click' in place
7. IV Glove is ready for use

**Note: If desired, add medical tape around the device for additional securement**

# IV Glove Strapless

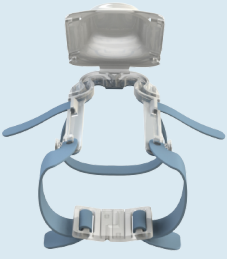

**Instructions  
for use**

*For ages 12 and under*

# IV GLOVE

## Assessment

- Follow hospital protocol for assessment
- Simply inspect through the transparent covered dome to ensure proper catheter placement (no redness, swelling, etc.)
- If further inspection is required, simply lift the transparent covered dome for closer inspection

## Removal

- 1.Open the transparent covered dome and remove the tubing from the S-Clip
- 2.Loosen the arm straps by removing the pins from the holes
- 3.Slide the IV Glove off the patient's arm, taking care to mind the IV insertion site
- 4.Remove the medical tape or Tegaderm
- 5.Remove the IV catheter

## Post-application

- IV Glove is made of 100% recyclable materials
- Dispose of the IV Glove per hospital protocol

**Scan for instruction video**

