



IV Glove 2.0 Adult

Pre-application: Clean the site per hospital protocol, and ensure the site is dry before applying the

**Instructions
for use**

For ages 12 and under

IV GLOVE

1. Follow standard IV insertion as per hospital protocol
2. Cover the insertion site with medical tape or Tegaderm
3. Open the transparent covered dome of the IV Glove by pulling up on the cover
4. Place 3 to 4 of patient's fingers through the IV Glove finger loop - according to patient and clinician preference
5. Fasten the wrist clip and adjust the blue stretchy wrist bands to ensure comfort and securement.
6. Ensure that you can slide your index and middle finger under the strap to ensure the wrist strap is not too tight
7. If necessary, adjust the finger loop straps to tighten by pulling the ends of the finger loop straps through the top of the device
8. Ensure capillary refill of less than 2 seconds distal to insertion site
9. Place catheter tubing into patented S-Clip design
10. Carefully close the protective dome, ensuring the pins 'click' in place
11. IV Glove is ready for use

IV Glove 2.0 Adult



**Instructions
for use**

For ages 12 and under

IV GLOVE

Assessment

- Follow hospital protocol for assessment
- Simply inspect through the transparent covered dome to ensure proper catheter placement (no redness, swelling, etc.)
- If further inspection is required, simply lift the transparent covered dome for closer inspection

Removal

- 1.Open the transparent covered dome and remove the tubing from the S-Clip
- 2.Loosen the wrist strap by removing the pin from the hole
- 3.Loosen the finger loop by removing the pin from the hole
- 4.Slide the IV Glove off the patient's hand, taking care to mind the IV insertion site
- 5.Remove the medical tape or Tegaderm
- 6.Remove the IV catheter

Post-application

- IV Glove is made of 100% recyclable materials
- Dispose of the IV Glove per hospital protocol

Scan for instruction video

