



IV Sleeve

Pre-application: Clean the site per hospital protocol, and ensure the site is dry before applying the

**Instructions
for use**

For ages 12 and under

IV Glove

1. Follow standard IV insertion as per hospital protocol
2. Cover the insertion site with medical tape or Tegaderm
3. Open the transparent covered dome of the IV Glove by pulling up on the cover
4. Place the device on the patient's arm and fasten the arm bands by pushing the pins through the appropriate holes, ensuring the device is secure
5. Place the remaining length of the strap back through the slit to ensure there are no loose straps hanging down
6. Cut excess straps if desired
7. Ensure that you can slide your index and middle finger under the strap to ensure the wrist strap is not too tight
8. Ensure capillary refill of less than 2 seconds distal to insertion site
9. Place catheter tubing into patented S-Clip design
10. Carefully close the protective dome, ensuring the pins 'click' in place
11. IV Glove is ready for use

IV Sleeve



**Instructions
for use**

For ages 12 and under

IV Glove

Assessment

- Follow hospital protocol for assessment
- Simply inspect through the transparent covered dome to ensure proper catheter placement (no redness, swelling, etc.)
- If further inspection is required, simply lift the transparent covered dome for closer inspection

Removal

- 1.Open the transparent covered dome and remove the tubing from the S-Clip
- 2.Loosen the arm straps by removing the pins from the holes
- 3.Slide the IV Glove off the patient's arm, taking care to mind the IV insertion site
- 4.Remove the medical tape or Tegaderm
- 5.Remove the IV catheter

Post-application

- IV Glove is made of 100% recyclable materials
- Dispose of the IV Glove per hospital protocol

Scan for instruction video

